HiaTech[®]

RGO

Ballon de calibrage du hiatus

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Hiatus calibration balloon

> **Laparoscopic treatment for Gastro-Œsophageal Reflux Disease (GERD)**

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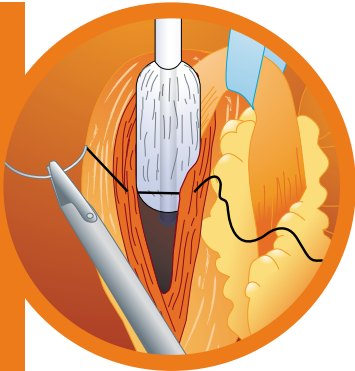
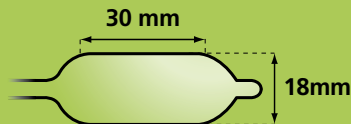
The **HiaTech®** calibrator has been designed to facilitate, reproduce and harmonize the laparoscopic calibration of the œsophageal hiatus through a number of features:

- Inflatable balloon 18 mm in diameter.
- Oblong shape that fits naturally between the crura without trauma to the œsophagus.
- Inflation with 9 ml of saline.
- Device protecting the balloon when the instrument is inserted and removed.
- Easy handling via a 5 mm trocar.
- Check valve to maintain pressure in the balloon.
- A bend at the abdominal end for perfect orientation whatever the position of the trocar.

Procedure :

- > The calibrator is inserted via a **5 mm trocar**.
- > After removal of the protective sleeve, the balloon is inflated with **9 ml of saline**, using a standard Luer-Lok™ syringe.
- > The balloon is inserted between the crura of the diaphragm.
- > The now calibrated hiatus can be closed by suturing together the crura, with the balloon in place if inserted via the retractor trocar.
- > The device is single-use but can be reinserted in the same patient following surgery to check the calibration.

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